

# Ketone Precision Xtra

- A simple, accurate cowside blood BHBA test used to identify ketotic cows and herds with ketosis/BHBA problems with fast results in less than 10 seconds.
- Specially designed end-fill/top-fill test strip with visual confirmation for fast and accurate results.
- Meter and blood ketone test strips (10 strips/box) equipment needed for testing system.
- \$3.30/test

## How to Use

1. Insert ketone test strip into the meter. The meter then “remembers” this lot of test strips and recognizes them automatically.
2. Obtain a drop of blood from the cow.
3. Apply a drop of blood from the syringe to the end of the test strip.
4. Wait 10 seconds for the meter to display the results. Cows that are 1.4 mmol/L or higher are considered positive for ketosis.



## Accuracy of the Blood BHBA Test

Three recent research studies reported as research abstracts from the AABP convention all gave very similar results and all confirm the excellent accuracy of the Precision Xtra™ blood ketone system for diagnosing ketosis in dairy cattle.

The results suggest that BHBA results from the hand-held meter are excellent – far better than the other cowside tests that are currently available.

**Ketone Test Strips \$32.95/10 strips**  
**Meter \$21.99**

**ANIMART®**

Dairy & Livestock Solutions  
800.255.1181 | animart.com